 Miscellaneous Ankle Supports - EN APPLICATION & CARE INSTRUCTIONS
 Tobilleras diversas - ES MODO DE EMPLEO Y CUIDADO DEL PRODUCTO
 Support de cheville divers - FR UTILISATION ET ENTRETIEN
 Verschiedene Sprunggelenkbandagen - DE ANWEISUNG ZUM ANLEGEN UND ZUR PFLEGE
 Diverse enkelsupports - NL INSTRUCTIES VOOR AANBRENGEN EN ONDERHOUD
 Διάφορα υποστηρίγματα αστραγάλου - GR ΟΔΗΓΙΕΣ ΕΦΑΡΜΟΓΗΣ ΚΑΙ ΦΡΟΝΤΙΔΑΣ
 Supporti vari per caviglia - IT ISTRUZIONI PER L'APPLICAZIONE E LA CURA DEL PRODOTTO
 Diverse ankelstøtter - DK INSTRUKTIONER I ANVENDELSE OG VEDLIGEHOLDELSE
 Soportes Diversos para o Tornozelo - PT INSTRUÇÕES DE UTILIZAÇÃO E CUIDADOS
 Różne uszywniacze kostki - PL STOSOWANIE I ZALECENIA DOTYCZĄCE OBCIĄŻENIA SIE Z PRODUKTEM
 Diverse ankelstöd - SE ANVISNINGAR FÖR ANVÄNDNING OCH SKÖTSEL
 Разнообразные голеностопные держатели - RU РУКОВОДСТВО ПО ИСПОЛЬЗОВАНИЮ И УХОДУ
 Různé kotníkové podpěry - CZ POKYNY PRO POUŽITÍ A OŠETŘOVÁNÍ
 各种踝部护具 - SC 使用与保养说明
 각종 발목 지지대 - KO 용도 및 관리방법
 各種踝部護具 - TC 繁體中文使用及護理說明
 Erlaiset nikkatuet - FI KÄYTTÖ- JA HOITO-OHJEET
 Razni steznici za kolena - SR UPUTSTVO ZA KORISTENJE I ODRŽAVANJE
 Razni steznici za koljena - HR PRIMJENA I ODRŽAVANJE
 دعامات مختلفة للكاحل - العربية تعليمات الاستخدام والعناية

 Peibagai Penahan Pergelangan Kaki - MS PANDUAN PEMAKAIAN & PENJAGAAN
 ส้นเท้าทั้งปีเจาะ - TH คำแนะนำในการใช้และดูแลรักษา
 Phụ Kiện Đỡ Mắt Cá Chân - VI HƯỚNG DẪN SỬ DỤNG VÀ BẢO DƯỠNG
 Разнообразни ортези за глезен - BG ИНСТРУКЦИИ ЗА УПОТРЕБА И ПОЧИСТВАНЕ
 Erinevad pahkluutoed - ET KASUTAMIS- JA HOOLDUSJUHEND
 Különböző bokaszorítók - HU ALKALMAZÁSI ÉS ÁPOLÁSI UTASÍTÁSOK
 Fjölbreyttir ökklastöðningar - IS LEIÐBEININGAR UM MEÐFERÐ OG NOTKUN
 Balstortozes potlēm - LV LIETOŠANAS OR KOPŠANAS NORĀDĪJUMI
 Ivairios kulno apsaugos - LT NAUDOJIMO IR PRIEŽIŪROS INSTRUKCIJOS
 Diverse støttebandasjer for ankel - NO BRUKS- OG VEDLIKEHOLDSANVISNING
 Diferite suporturi pentru gleznă - RO INSTRUCȚIUNI DE APLICARE ȘI ÎNTREȚINERE
 Ďalšie bandáže na členok - SK POKYNY NA ZAKLADANIE A STAROSTLIVOSŤ
 Raznorazne opore za gležnjei - SL NAVODILA ZA UPORABO IN NEGGO
 Çeşitli Ayak Bileği Destekleri - TR UYGULAMA VE BAKIM TALİMATLARI

Ankle Support 41600B, 41640, 41641, 41642, 41643, 41643, 41644, 41645B
Wraparound Ankle Support 43637
Adjustable Ankle Support 42037, 42038B, 42039B
Adjustable Ankle Stabilizer 45677, 45678B, 45679B

Intended Use:
Ankle Support- Provide support to weak, injured, or arthritic ankle joints.
Ankle Stabilizer- Provide support and protection to weak, injured, or arthritic ankles.
How to Apply: (41600B, 41640, 41641, 41642, 41643, 41643, 41644, 41645B)
1. Place thumbs inside support and gather the material
2. Slip on to foot
3. Align heel opening with heel
How to Apply: (43637)
1. Place heel in round opening.
2. Fasten adjustable straps around ankle.
How to Apply: (42037, 42038B, 42039B)
1. Slip onto foot.
2. Align heel opening with heel.
3. Secure upper strap.
4. Criss-cross tension straps.
5. Wrap the straps around the back of the ankle and secure.
How to Apply: (45677, 45678B, 45679B)
1. Slip onto foot.
2. Align heel opening with heel.
3. Secure upper strap.
4. Criss-cross tension straps.
5. Wrap the straps around the back of the ankle and secure.
Some products constructed of a neoprene blend may cause skin irritation.
If rash develops or pain persists, discontinue use and consult a physician.
CARE: Hand wash in cold water with mild detergent. Do not use bleach or fabric softener. Rinse thoroughly and air dry.
Notice: Should any serious health related incident occur with the use of this device, please notify the manufacturer identified on the label and the competent authorities of your country.

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