Bandages & Wraps - EN APPLICATION & CARE INSTRUCTIONS
Vendas - ES MODO DE EMPLEO Y CUIDADO DEL PRODUCTO
Bandes et bandages - FR UTILISATION ET ENTRETIEN
Binden und Bandagen - DE ANWEISUNG ZUM ANLEGEN UND ZUR PFLEGE
Bandages en wraps - NL INSTRUCTIES VOOR AANBRENGEN EN ONDERHOUD
Επίδεσμοι & ελαστικά περιτυλίματα - GR ΟΔΗΓΙΕΣ ΕΦΑΡΜΟΓΗΣ ΚΑΙ ΦΡΟΝΤΙΔΑΣ
Medicazioni e avvolgimenti - IT ISTRUZIONI PER L'APPLICAZIONE E LA CURA DEL PRODOTTO
Bandager og bindinger - DK INSTRUKTIONER I ANVENDELSE OG VEDLIGEHOLDELSE
Faixas e Ligaduras - PT INSTRUÇÕES DE UTILIZAÇÃO E CUIDADOS
Bandáže zwykłe i elastyczne - PL STOSOWANIE I ZALECENIA DOTYCZĄCE OBCHODZENIA SIĘ Z PRODUKTEM
Förband och bindor - SE ANVISNINGAR FÖR ANVÄNDNING OCH SKÖTSEL
Многоразовые бандажы и разовые повязки - RU РУКОВОДСТВО ПО ИСПОЛЬЗОВАНИЮ И УХОДУ
Obinadla a zábalý - CZ POKYNY PRO POUŽITÍ A OŠETŘOVÁNÍ
绷带和包扎带 - SC - 简体中文使用与保养说明
반창고 및 붕대 - KO 적용 및 주의사항
繃帶和包紮帶 - TC - 繁體中文使用及護理說明
Tuki- ja joustosidokset - FI KÄYTTÖ- JA HOITO-OHJEET
Bandáže i zavoji - SR UPUTSTVO ZA KORIŠTENJE I ODRŽAVANJE
Bandáže trake i zavoji - HR PRIMJENA I ODRŽAVANJE
الضمادات واللفافات - العربية تعليمات الاستخدام والعناية

Pembalut & Balut - MS PANDUAN PEMAKAIAN & PENJAGAAN
ผ้าพัน - TH คำแนะนำในการใช้และดูแลรักษา
Băng y tế và bọc - VI HƯỚNG DẪN SỬ DỤNG VÀ BẢO DƯỠNG

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Intended Use: Helps provide compression and support to body parts
How to Apply: (6152, 6152ML, 6153, 6153ML, 6154, 6154ML, 6156, 6156ML, 350101, 350102, 350103, 350104)
1. Wash and dry area to be wrapped.
2. Begin wrapping, partially overlapping previous layer.
3. Secure Velcro® end or clips to hold in place.
4. Avoid wrapping too tight.

How to Apply: (6263, 6264, 6265)

- Wrap desired area, partially overlapping previous layer.
 - Sticks to self, does not need clips.
 - Be sure not to wrap body part too tight.
- How to Apply:** (3471, 4590, 4591, 6183, 6183ML, 6285, 6285ML)
1. Begin wrapping, partially overlapping previous layer.
2. Secure tab to hold in place.
3. Avoid wrapping too tight.

CARE (6152, 6152ML, 6153, 6153ML, 6154, 6154ML, 6156, 6156ML, 350101, 350102, 350103, 350104): Hand wash in warm water with mild detergent. Rinse thoroughly, smooth out all wrinkles and air dry.

CARE (3471, 4590, 4591, 6183, 6183ML, 6285, 6285ML): Hand wash in cold water with mild detergent. Do not use bleach or fabric softener. Rinse thoroughly and air dry.

CAUTION (6152, 6152ML, 6153, 6153ML, 6154, 6154ML, 6156, 6156ML, 350101, 350102, 350103, 350104): This is an elastic product and care should be taken not to constrict blood flow. Recommended for daytime wear only.

****CAUTION:** This product contains natural rubber latex which may cause allergic reactions. Some products constructed of a neoprene blend may cause skin irritation.

If rash develops or pain persists, discontinue use and consult a physician.

Notice: Should any serious health related incident occur with the use of this device, please notify the manufacturer identified on the label and the competent authorities of your country.

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