Mueller [®]	Contents: <p>NASAL PLUGS NASAL SPONGES</p>
ML1 12208 - 2.20	

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Intended use: Aids in the treatment of nose bleeds.

Nasal Plugs (191221, 191222)
Instructions for Use: NOTE: When assisting someone, protective gloves are recommended.

- Grasp end of nasal plug.
- Gently insert nasal plug into first inch of nose.
- When bleeding has stopped, grasp end of nasal plug and gently withdraw from the nose.

Nasal Sponges (191201, 191202)
Instructions for Use: NOTE: When assisting someone, protective gloves are recommended.

- INSERTION: Grasp arched end of pack. Cover leading edge with antibiotic lubricant and gently insert.
- EXPANSION: Physicians typically leave pack in site up to 3 days. If pack is not fully expanded after 30 seconds, hydrate with 5-10 ml saline or sterile water.
- REMOVAL: Saturate with sterile water or saline. Wait 10 minutes before removal. Grasp pack with forceps and gently withdraw.

How to Apply: (140501, 440501)

- Gently place nose guard on face so that the vinyl foam padding around the cheek and forehead areas are in a comfortable position.
- Gently stretch the elastic harness over head to hold nose guard securely in place.
- Adjust straps as needed.
- Care: Wipe with a soft damp cloth.

Not made with natural rubber latex.

If rash develops or pain persists, discontinue use and consult a physician.

Notice: Should any serious health related incident occur with the use of this device, please notify the manufacturer identified on the label and the competent authorities of your country.

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©2020 Mueller Sports Medicine, Inc.