## Contents: 1 SUPPORT

MLI66117B\_2.22

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### ADJUSTABLE POSTURE SUPPORT - EN 66118ML and 66119BML

**Intended Use:** Posture Support - Provides light support to the upper back to encourage proper posture

**How to Apply** (66118ML and 66119BML)

- Loosen the straps by unfastening and adjusting the red hook tabs then refastening them to the shoulder pads.
- Hold the Posture Support with the Mueller logo facing outward and slip on like a backpack. The elastic straps will stretch over each shoulder.
- While holding correct posture, adjust the fit. Unfasten the red hook tabs to tighten or loosen straps and refasten when support feels snug. You may need to remove the product in order to adjust.
- Start by wearing for short durations (30 minutes). Increase duration as your body adapts.

**CARE:** Hand wash in cool water with mild detergent. Do not use bleach or fabric softener. Rinse thoroughly and air dry.

If rash develops or pain persists, discontinue use and consult a physician.

**Notice:** Should any serious health related incident occur with the use of this device, please notify the manufacturer identified on the label and the competent authorities of your country.

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©2021 Mueller Sports Medicine, Inc.