<b>Ankle Supports - EN</b>
APPLICATION & CARE INSTRUCTIONS
<b>Tobileras - ES</b>
INSTRUCCIONES PARA SU COLOCACION Y CUIDADO
<b>Supports de cheville - FR</b>
UTILISATION ET ENTRETIEN
<b>Sprunggelenkbandagen - DE</b>
ANWEISUNG ZUM ANLEGEN UND ZUR PFLEGE
<b>Enkelsteunen - NL</b>
INSTRUCTIES VOOR AANBRENGEN EN ONDERHOUD
<b>Υποστηρίγματα στραγάλου - GR</b>
ΟΔΗΓΙΕΣ ΕΦΑΡΜΟΓΗΣ ΚΑΙ ΦΡΟΝΤΙΔΑΣ
<b>Supporti per caviglia - IT</b>
ISTRUZIONI PER L'APPLICAZIONE E LA CURA DEL PRODOTTO
<b>Ankelstøtter - DK</b>
INSTRUKTIONER I ANVENDELSE OG VEDLIGEHOLDELSE
<b>Supportes para tornozelo - PT</b>
INSTRUÇÕES DE UTILIZAÇÃO E CUIDADOS
<b>Stabilizatori kostki - PL</b>
STOSOWANIE I ZALECENIA DOTYCZĄCE OBCHODZENIA SIĘ Z PRODUKTEM
<b>Ankelskydd - SE</b>
ANVISNINGAR FÖR ANVÄNDNING OCH SKÖTSEL
<b>Голеностопные держатели - RU</b>
РУКОВОДСТВО ПО ИСПОЛЬЗОВАНИЮ И УХОДУ

<b>Podpěry kotníku - CZ</b>
POKYNY PRO POUŽITÍ A OŠETŘOVÁNÍ
<b>护踝 - SC - 简体中文</b>
使用与保养说明
<b>발목 보호대 - KO</b>
적용 및 주의사항
<b>護踝 - TC - 繁體中文</b>
使用及護理說明
<b>Nilkkatuet - FI</b>
KÄYTTÖ- JA HOITO-OHJEET

<b>Steznici za skočni zglob - SR</b>
UPUTSTVO ZA KORIŠTENJE I ODRŽAVANJE

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- Slide foot into support.
- Support can be worn over sock or over bare foot.
- How to Apply:** (965SM, 965MD, 965LG, 965XL)

- Slide foot into support.
- Wrap around inner strap to desired compression.
- Wrap around outer strap to desired compression.

**How to Apply:** (4547, 4547EU, 6511, 6511C, 6511ML)

- Place heel in opening.
- Fasten strap over arch.
- Fasten strap around ankle.
- Cross-cross tension straps around ankle.
- Adjust as needed for a comfortable fit.

**How to Apply:** (40007, 40008, 40009)

- Slide support onto foot.
- Center support on ankle.
- Adjust as needed for a comfortable fit.

**How to Apply:** (65517ML)

- Slide foot into support.
- Wrap lower strap around back of ankle and fasten tab on the front
- Wrap upper strap around back of leg and fasten tab on the front
- How to Apply:** (65557ML)

**How to Apply:** (965SM, 965MD, 965LG, 965XL)

- Place heel in opening.
- Fasten strap over arch.
- Fasten strap around ankle.
- Cross-cross tension straps around ankle.
- Adjust as needed for a comfortable fit.

**How to Apply:** (40007, 40008, 40009)

- Slide support onto foot.
- Center support on ankle.
- Adjust as needed for a comfortable fit.

**How to Apply:** (65517ML)

- Slide foot into knick section of the support with care tag against the shin
- Wrap lower strap around back of ankle and fasten tab on the front
- Wrap upper strap around back of leg and fasten tab on the front
- How to Apply:** (65557ML)

**How to Apply:** (965SM, 965MD, 965LG, 965XL)

- Slide loop onto foot so silicone is facing towards your ankle
- Wrap the strap behind the ankle with silicone against the skin or sock
- Wrap the strap over and then under the foot
- Continue wrapping around the front of the ankle, secure tab
- Adjust to desired compression level

\*\*\***CAUTION:** This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Some products constructed of a neoprene blend may cause skin irritation.

If rash develops or pain persists, discontinue use and consult a physician.

**CARE:** Hand wash in cold water with mild detergent. Do not use bleach or fabric softener. Rinse thoroughly and air dry.

Notice: Should any serious health related incident occur with the use of this device, please notify the manufacturer identified on the label and the competent authorities of your country.

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